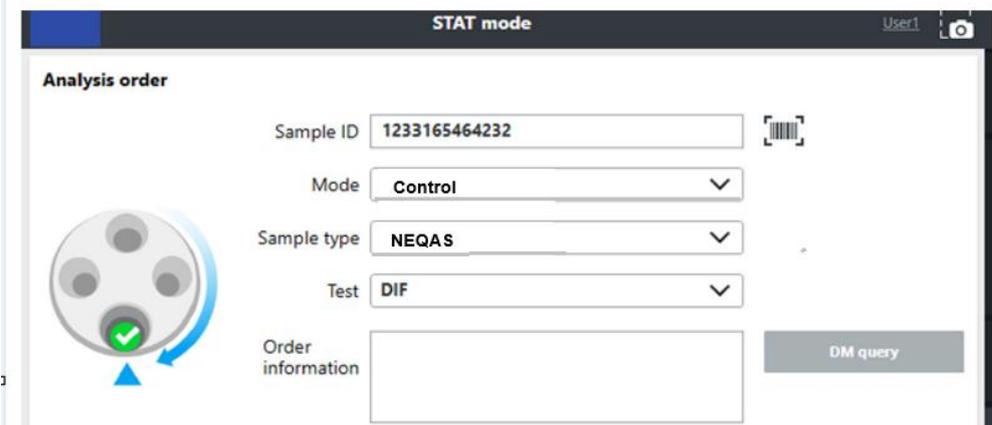



UK NEQAS Manufacturer’s Proforma Template

Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

	
Title	Instructions for Running UK NEQAS FB, RE and DL EQA Specimens on the HORIBA Yumizen H1500/H2500, v2, instruments
Analysis Instructions	<p>A specific sample type called NEQAS will be created when the analyser is installed. If not present, please contact HORIBA Medical immediately.</p> <p>Prepare your control blood according to the specific instructions detailed in the control blood package insert.</p> <p>❖ Processing FB and DL EQA specimens</p> <ol style="list-style-type: none"> 1. Select STAT mode 2. Enter the sample ID either manually or using the barcode reader 3. Change mode to Control. 4. Change sample type to NEQAS. <p>DO NOT use the DM enquiry option</p>  <ol style="list-style-type: none"> 5. Mix the specimen 6. Place specimen in relevant position of barrel 7. Select Start Analysis <p>❖ Processing RE EQA specimens</p> <ol style="list-style-type: none"> 1. Select STAT mode 2. Enter the sample ID either manually or using the barcode reader. 3. Change sample type to NEQAS. Do not change mode. 4. Select RET as test. DO NOT use the DM enquiry option.

UK NEQAS Manufacturer's Proforma Template

Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

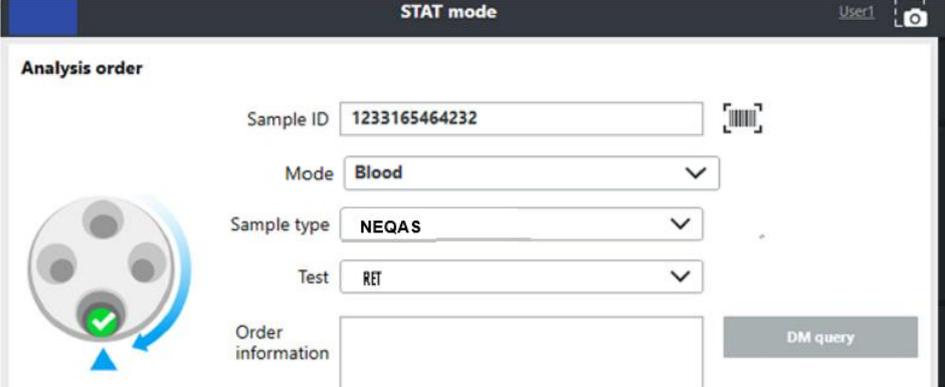	 <p>5. Mix the sample 6. Place sample in relevant position of barrel 7. Select Start Analysis</p>
<p>Additional Notes:</p>	<p>~</p>
<p>Contact Information</p>	<p>Technical Support Team Email Andrew.fisher@horiba.com Telephone Number 01604542670</p>
<p>Authorised by</p>	<p>Name and Signature Andrew S Fisher </p>
<p>Document Control</p>	<ul style="list-style-type: none"> ○ ECD Haem 17 ○ Version number: 2 ○ Authorisation date: